

Safety-Lancet – The new generation

Safe and easy sampling with increased patient comfort



Safety for both user and patient

- The sterile needle or blade is safely concealed in the lancet body before and after use, preventing the risk of needlestick injury
- Automatic locking feature eliminates re-use and cross-infection
- Secure firing button prevents accidental activation


Easy sampling

- Primed and ready for use
- Ridged lancet body ensures safe grip
- Small contact face for precise puncturing

Patient comfort

- Smooth siliconised blades and triple-sharpened needlepoints to minimise pain
- Defined penetration depths for optimal sampling from adults and children
- Controlled penetration depth to eliminate the risk of bone injury

Safety-Lancet – The new generation

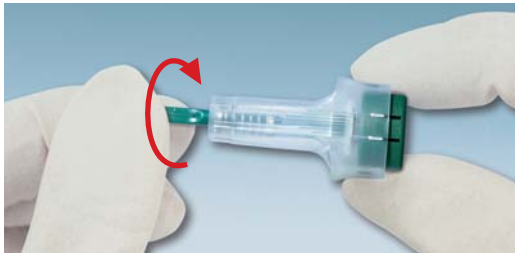

- 1 Twist the cap until it separates from the Safety-Lancet.



- 2 Press the Safety-Lancet firmly against the chosen and cleaned puncture site and press the firing button.



- 3 Dispose of the Safety-Lancet into a sharps container.

Ordering information

Order No.	Description	Size	Penetration depth	Packaging
85.1015	Safety-Lancet <i>Mini</i>	28 G	1.6 mm	200/inner box 1,200/case
85.1016	Safety-Lancet <i>Normal</i>	21 G	1.8 mm	200/inner box 1,200/case
85.1017	Safety-Lancet <i>Extra</i>	18 G	1.8 mm	200/inner box 1,200/case
85.1018	Safety-Lancet <i>Super</i>	Blade 1.5 mm	1.6 mm	200/inner box 1,200/case
85.1019 (For infants' heels)	Safety-Lancet <i>Neonatal</i>	Blade 1.5 mm	1.2 mm	200/inner box 1,200/case

Microvette® capillary blood collection tubes complement this product range



Microvette® 100/200
Blood collection with
End-to-End capillary



Microvette® 300/500
Blood collection with
collection rim

More information is available on request.

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 3050
Fax +49 2293 305-122
info@sarstedt.com
www.sarstedt.com